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English

This item complies with the Product Safety Act (ProdSG).

1. Intended Use

The Oliver positioning aid (half-roll) is designed for use in therapy, massage, and relaxation. It is used for optimal positioning and pressure relief of various body parts (e.g., under the knees, neck, or back) in supine or side-lying positions.

2. Safety Information

- **Not a toy:** The positioning aid is a therapy and sports device. Children should only use the product under adult supervision.
- **Standing prohibited:** Do not stand on the half-roll. Due to the curved shape and material properties, there is an extreme risk of falling.
- **Check before use:** Regularly inspect the roll for damage to the cover or the foam core. Damaged products should no longer be used for hygiene or safety reasons.
- **What can happen:** The roll slipping on very smooth surfaces can lead to sudden changes in position. Ensure stable placement on a suitable surface (e.g., massage table or gymnastics mat).

3. Maintenance and Storage

- **Cleaning:** The half-roller can be wiped down with a damp cloth and mild soapy water. Do not use harsh or alcohol-based cleaners to avoid damaging the surface material. Follow specific disinfection instructions for professional use if applicable.
- **Storage:** Store in a dry place protected from direct sunlight.

4. Disposal

The item can be disposed of with household waste or at local recycling centers.

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